# FIT KIT for WalkOn Reaction

Quality for life

# FIT KIT for WalkOn Reaction

<b>DE</b>	Gebrauchshinweise .....	4
<b>EN</b>	Instruction folder .....	6
<b>IT</b>	Vedere allegato .....	8
<b>ES</b>	Carpeta de instrucciones .....	10
<b>NL</b>	Instructiefolder .....	12
<b>NO</b>	Bruksanvisning .....	14
<b>SV</b>	Instruktionsfolder .....	16
<b>DA</b>	Brugsanvisning .....	18
<b>CS</b>	Pokyny pro použití .....	20
<b>FI</b>	Ohjekansio .....	22

Best.-Nr. 28T3

## WalkOn FIT KIT für 28U24 WalkOn Reaction

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Art. no. 28T3

## WalkOn FIT KIT for 28U24 WalkOn Reaction

This FIT KIT is testing equipment designed to facilitate the correct choice of a WalkOn AFO (ankle foot orthoses). The objective is to assess if a WalkOn Reaction is appropriate for the patient. The test orthoses are intended to evaluate whether the size is correct and the desired effect is achieved.

The FIT KIT is suited for WalkOn Reaction, article number 28U24. (The WalkOn Reaction is designed with a rubber sole, which is attached to the footplate and can be trimmed, making it easy to customise with the use of a simple pair of scissors).

### **The FIT KIT includes eight orthoses:**

One orthosis is included for each side in the shoe sizes 36, 39, 42 and 45 respectively.

## How to use the FIT KIT?

---

- ▶ Choose a suitable product in terms of size and side for your patient.
  - ▶ Place the AFO in the patient's shoe, preferably under any removable insole.
  - ▶ Apply the shoe and product onto the patient.
  - ▶ Let the patient walk a short distance and evaluate the effectiveness of the AFO.
  - ▶ If the desired effectiveness is not yet achieved, you can work with the enclosed heel or forefoot elevating elements in order to obtain better results. In case of supination or inversion of the forefoot, a lateral pronation cable which is included may be advisable in addition.
  - ▶ If the result is positive, fit the patient with the appropriate size of a WalkOn Reaction.
- 

If the patient needs an orthosis with less support, choose a classic 28U11 WalkOn or a 28U23 WalkOn Trimable (a classic WalkOn but with a rubber sole attached to the footplate), or a more flexible orthosis such as the 28U22 WalkOn Flex.

(WalkOn / WalkOn Trimable FIT KIT, article number 28T1N; WalkOn Flex FIT KIT, article number 28T2)

### Note

Since the test model is often not fully adapted to the patient, some pressure may be experienced at the medial malleolus during use. This is remedied by an individually adapted orthosis, grinding or cutting according to the fitting instructions, (see the product instructions for use).

Art. no	Side	Shoe Size	Height
28U24=L/R 36-39	Left/Right	36-39	35 cm / 13,8 inch
28U24=L/R 39-42	Left/Right	39-42	36 cm / 14,2 inch
28U24=L/R 42-45	Left/Right	42-45	37,5 cm / 14,8 inch
28U24=L/R 45-48	Left/Right	45-48	39 cm / 15,4 inch

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Otto Bock HealthCare GmbH