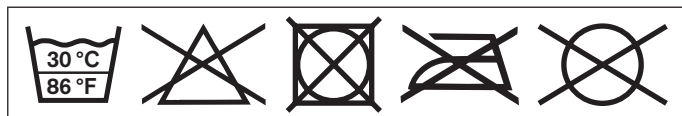
50S6 Achillo Sensa

DE	Gebrauchsanweisung	3
EN	Instructions for use	6
FR	Instructions d'utilisation	10
IT	Istruzioni per l'uso	14
ES	Instrucciones de uso	18
PT	Manual de utilização	22
NL	Gebruiksaanwijzing	25
SV	Bruksanvisning	29
DA	Brugsanvisning	33
NO	Bruksanvisning	36
FI	Käyttöohje	40
PL	Instrukcja użytkowania	44
HU	Használati utasítás	48
CS	Návod k použití	51
HR	Upute za uporabu	55
SK	Návod na používanie	59
BG	Инструкция за употреба	62
TR	Kullanma talimatı	66
EL	Οδηγίες χρήσης	70
RU	Руководство по применению	74
ZH	使用说明书	78

	Seite/Side	Größe/Size	Umfang oberhalb des Knöchels / Circumference above the ankle	
			cm	inch
	L/R	XS	17-19	6.7-7.5
	L/R	S	19-21	7.5-8.3
	L/R	M	21-23	8.3-9.1
	L/R	L	23-25	9.1-9.8
	L/R	XL	25-27	9.8-10.6

Material	Gestrick/Knit Fabric: PA, EL
	Frotteebezüge/Terry Cloth Covers: Baumwolle/Cotton, PES
	Pelotte/Pad: viskoelastisches Technogel®/viscoelastic Technogel®

INFORMATION

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INFORMATION

Date of last update: 2020-03-31

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.

- | |
|---|
| <ul style="list-style-type: none">▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.▶ Please keep this document for your records. |
|---|

These instructions for use provide important information on the fitting and application of the 50S6 Achillo Sensa Achilles tendon support.

2 Intended use

2.1 Indications for use

The support is intended **exclusively** for fittings of the lower limbs and **exclusively** for contact with intact skin.

The support must be used according to the indication.

2.2 Indications

- Postoperative and posttraumatic irritation
- Achillodynia
- Subsequent treatment of ruptured Achilles tendon

Indications must be determined by the physician.

2.3 Contraindications

2.3.1 Absolute Contraindications

Not known.

2.3.2 Relative Contraindications

The following indications require consultation with a physician: skin diseases/injuries, inflammation, prominent scars that are swollen, reddening and hyperthermia of the fitted limb; pronounced varicose veins, especially with return flow impairment, lymphatic flow disorders, including unclear soft tissue swelling distant to the body area to which the support will be applied; sensory and circulatory disorders in the foot area, e.g. in case of diabetic neuropathy.

2.4 Effects

The form-knitted support with integrated shape-retaining pad combined with the heel wedge provides relief for the Achilles tendon. The support promotes sensorimotor functions and thereby activates muscular stabilisation. The additional heel wedge is used on the sound foot to compensate for different leg lengths.

3 Safety

3.1 Explanation of warning symbols

CAUTION Warning regarding possible risks of accident or injury.

NOTICE Warning regarding possible technical damage.

3.2 General safety instructions

CAUTION

Contact with heat, embers or fire

Risk of injury (such as burns) and risk of product damage.

- ▶ Keep the product away from open flames, embers and other sources of heat.

CAUTION

Reuse on other persons and improper cleaning

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ The product may be used by one person only.
- ▶ Clean the product regularly.

NOTICE

Contact with oils, salves, lotions or other products that contain oils or acids

Insufficient stabilisation due to loss of material functionality

- ▶ Do not expose the product to oils, salves, lotions or other products that contain oils or acids.

4 Handling

INFORMATION

- ▶ The daily duration of use and period of application are generally determined by the physician.
- ▶ The initial fitting and application of the product must be carried out by qualified personnel.
- ▶ Instruct the patient in the handling and care of the product.
- ▶ Instruct the patient to see a physician immediately if any exceptional changes are noted (e.g. worsening of the complaint).

4.1 Size Selection

- 1) Measure the circumference above the ankle.
- 2) Determine the size of the support (see size chart).

4.2 Application

CAUTION

Incorrect or excessively tight application

Skin irritation due to overheating, local pressure points due to tight application to the body

- ▶ Ensure that the product is applied properly and fits correctly.
- ▶ When in doubt, do not continue to use the product if skin irritation occurs.

NOTICE

Use of a worn or damaged product

Limited effectiveness

- ▶ Before each use, check the product for functional reliability and for possible wear or damage.
- ▶ Do not continue using a product that is no longer functional, or that is worn or damaged.

- 1) Fold the support down enough so the heel can be grasped (see fig. 1).
- 2) Pull the support over the foot up to the heel (see fig. 2).
- 3) Unfold the section of the support that was folded over, i. e. unroll it upwards (see fig. 3).
- 4) Pull up the support so that the openings in the pads are over the Achilles tendon (see fig. 4).
- 5) When the patient is given the support, it must be checked for proper fit and positioning.

4.3 Cleaning

Clean the support regularly:

- 1) **Recommendation:** Use a laundry bag or net.
- 2) Wash the support in warm water at 30 °C with standard mild detergent. Do not use fabric softener. Rinse thoroughly.
- 3) Allow to air dry. Do not expose to direct heat (e.g. sunshine, stove or radiator).

5 Disposal

Dispose of the product in accordance with national regulations.

6 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

6.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

6.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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