Lörracher Bewegungsschiene

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


Quality for life


Lörracher

Bewegungsschiene 28L8


DE	Elterninformation	6
EN	Information for the Parents	8
FR	Informations aux parents	10
IT	Informazioni per i genitori	12
ES	Información para los padres	14
PT	Informação para os pais	16
NL	Informatie voor de ouders	18
SV	Föräldrarinformation	20
DA	Forældreinformation	22
PL	Informacja dla rodziców	24
CS	Informace pro rodiče	26
HU	Információ szülők részére	28
TR	Ebeveyn Bilgilendirme Broşürü	30
EL	Πληροφορίες για τους γονείς	32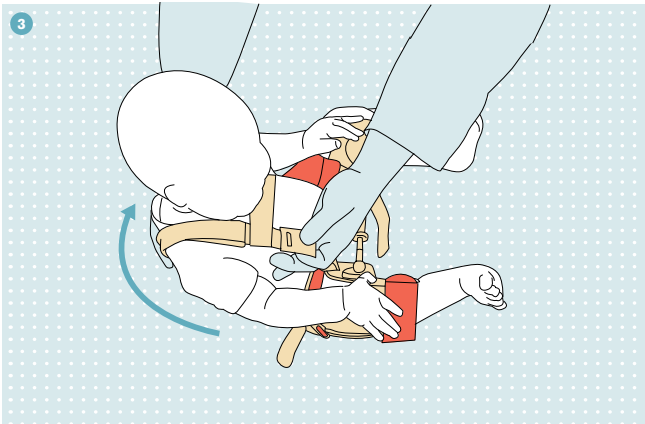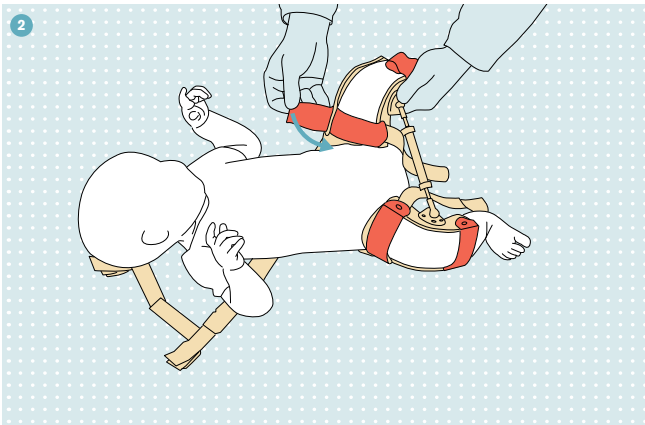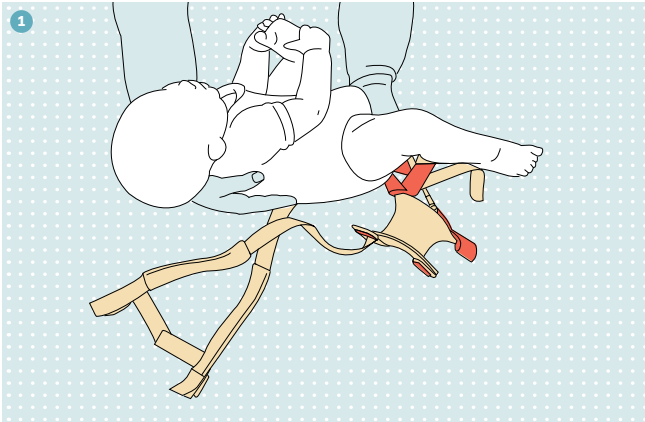


4

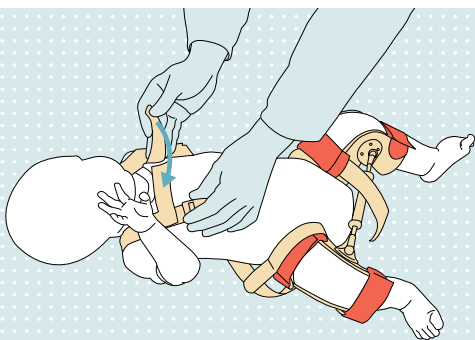5

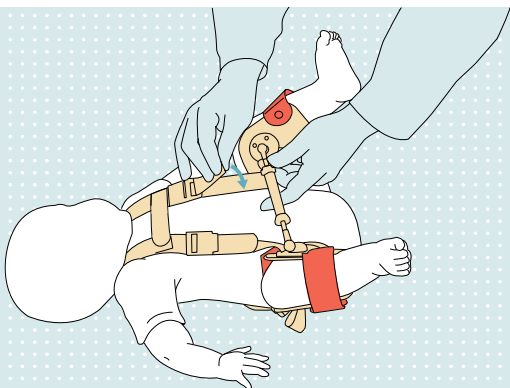6

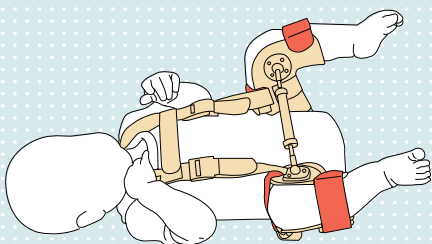Lörracher Bewegungsschiene 28L8

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Lörrach Hip Abduction Orthosis 28L8

By Dr. Behrens

For treating congenital hip dysplasia the therapeutic position combining hip flexion and abduction has proved to be effective. It corresponds to the natural position assumed by a child before birth.

Thus, when developing the Lörrach Hip Abduction Orthosis emphasis was placed to more flexion and moderate abduction. Jointed attachment of the thigh supports permits adjustable movement, and in this way promotes the development of the hip joint. The Lörrach Hip Abduction Orthosis was proven itself effective in clinic and practice.

Fitting of the orthosis provided by the doctor and the orthotist, includes adjustment of the flexion position through the harness and abduction to be adjusted by the threaded spanner along with the checking of fit and function.

[This information explains the use of the orthosis, since your cooperation as parents is important to the success of treatment.](#)

Please, follow the instructions given by your doctor, and as to the adjustment, do not make changes on your own responsibility.

Diapers can be changed with the orthosis applied, if this is to be worn under the clothing. Cotton stockinettes (available in the orthopaedic specialist enterprise) have proven to be effective. They are pulled over the legs and are to be changed regularly.

Normally, the orthosis may remain in place for about 23 hours a day allowing one hour for care (the doctor's prescription should be given priority).

[To apply or remove the orthosis, it is necessary to use only the anterior straps of the shoulder harness and the hook and pile closures of the leg supports.](#)

For donning the orthosis including harness, open its closures and position the child on the crosswise arranged straps (fig. 1).

Place the child's flexed legs in the thigh supports and secure the hook and pile straps (fig. 2).

Pull the shoulder straps up and over the infant's shoulders (fig. 3) to the front and fasten the chest cross strap (fig. 4).

Pull the anterior straps through the clip closures up to the mark determined by the doctor and close by pressure (fig. 5).

Most children quickly adjust to the orthosis (fig. 6). However, if your child cries continuously, the doctor should be consulted.

With a foam pillow or bath towel folded together, the child may comfortably sleep on his/her stomach.

During regular examinations the doctor will examine the hip joints along with fit and function of the orthosis. Corrections regarding adjustment as well as alterations due to the growth of the child will be provided as well on this occasion.

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